

Syringe Pump Extension Set

30" Minibore extension set with slide clamp,
male luer-lock, and female luer-lock

FOR PARENTERAL FLUIDS

CAUTIONS:

- Do not use if package is damaged and/or protection caps are detached. Keep dry. Store at controlled room temperature. Keep away from sunlight.
- Use aseptic technique.
- Single use only.

MORE INFORMATION:

- Length is approximate
- Approximate contained volume 0.5 mL
- Gravity flow rate 1,176 mL/hr
- Sterile and nonpyrogenic fluid path
- Fluid Path Material: PVC (Not made with DEHP) / ABS (male luer) / PE (end caps) / PC (female luer)

CONTRAINDICATIONS:

- Do Not Use for light sensitive drugs unless indicated. Some drugs may be incompatible with tubing materials and components check drug labeling and facility protocol.

PRIMING:

Priming must be completed before connecting to patient.

- Attach syringe, prime with medication, and ensure all air is removed
- Clamp tubing
- Swab patient's access site or other luer connection before connecting to patient per facility protocol
- Remove end cap and attach to patient luer

SET REMOVAL:

- Stop infusion. Close slide clamp, disconnect the set from IV access and container in accordance with facility protocol. Replace per facility protocol or CDC guidelines. Discard biohazard waste according to facility protocol.

Manufactured For:
Fresenius Kabi AG
Else-Kröner-Str.1
61352 Bad Homburg, Germany
www.fresenius-kabi.com

Distributed in USA by:
Fresenius Kabi USA, LLC
Three Corporate Drive
Lake Zurich, Illinois 60047
www.fresenius-kabi.us
Tel.: 1-800-933-6925

N. Code No. : HR/Drugs 737-B(H)

10.5520.00

10.5950.01