

Long Ext-2 Y 3-Way Stopcock Set

40" Macrobore extension set with lipid resistant 3-way macrobore stopcock with rotating male luer-lock, 2 K-Zero needleless Y-sites, and male luer-lock with retractable collar

FOR PARENTERAL FLUIDS

CAUTIONS:

- Do not use if package is damaged and/or protection caps are detached. Keep dry. Store at controlled room temperature. Keep away from sunlight.
- Use aseptic technique.
- Single use only.

MORE INFORMATION:

- Length is approximate
- Approximate contained volume 6.3 mL
- Gravity flow rate 12,600 mL/hr
- For Gravity use, flow rate is regulated by roller clamp
- Sterile and nonpyrogenic fluid path
- Fluid Path Material: PVC (Not made with DEHP) / ABS (retractable spin luer) / PE (caps & 3-way stopcock) / PC (3-way stopcock & female luer)

CONTRAINDICATIONS:

- Materials of the the gravity administration set does not provide light protection.

PRIMING:

Priming must be completed before connecting to patient.

- Connect to primary set
- Tighten all connections
- Before use, ensure that the 3-way stopcock is properly primed to remove any air bubbles
- Replace white cap per hospital policy
- Clamp set
- Swab patient's access site or other luer connection before connecting to patient per facility protocol
- Remove end cap and attach to patient luer
- To use stopcock, align arrows on handles with desired flow path

ATTACHING TO NEEDLELESS CONNECTORS:

- Swab surface of the needleless connector prior to and after every use according to CDC guidelines or facility protocol. Use luer lock connections only. If using luer slip, do not leave unattended.
- Do not use sharp needle or blunt cannula with the needleless connectors.

SET REMOVAL:

- Stop infusion. Close slide clamp, disconnect the set from IV access and container in accordance with facility protocol. Replace per facility protocol or CDC guidelines. Discard biohazard waste according to facility protocol.

Manufactured For:
Fresenius Kabi AG
Else-Kröner-Str.1
61352 Bad Homburg, Germany
www.fresenius-kabi.com

Distributed in USA by:
Fresenius Kabi USA, LLC
Three Corporate Drive
Lake Zurich, Illinois 60047
www.fresenius-kabi.us
Tel.: 1-800-933-6925

N. Code No. : HR/Drugs 737-B(H)

10.5520.00

10.5959.01